



To enter setup mode:
(Pace must be in Pause mode)

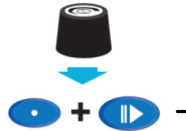


Release all 3 buttons

SETUP

Setting up the stimulator for dropped foot

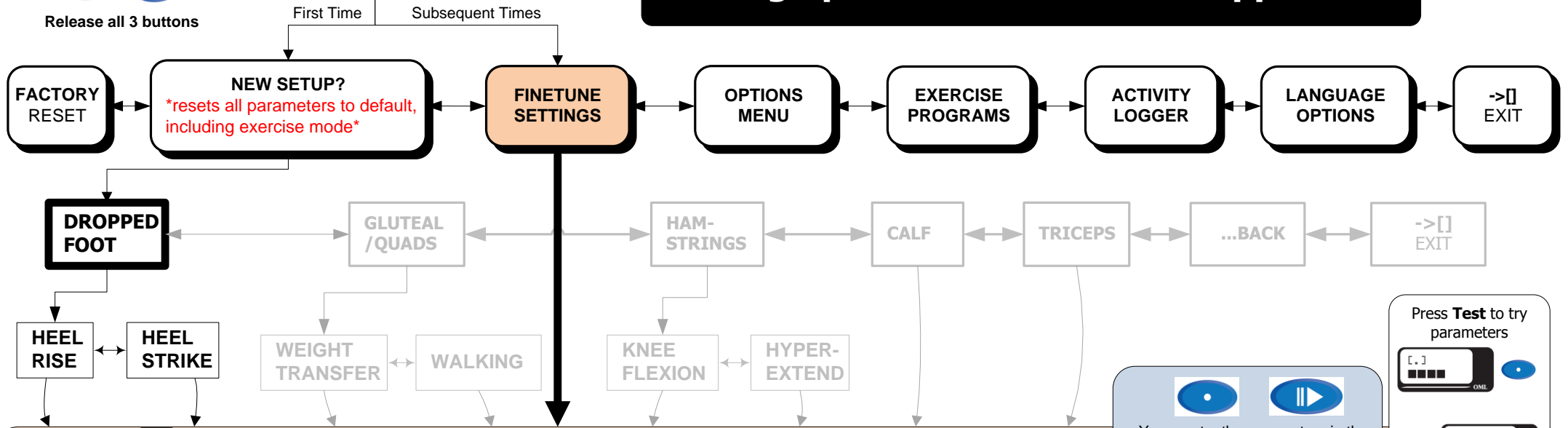

FINE TUNE

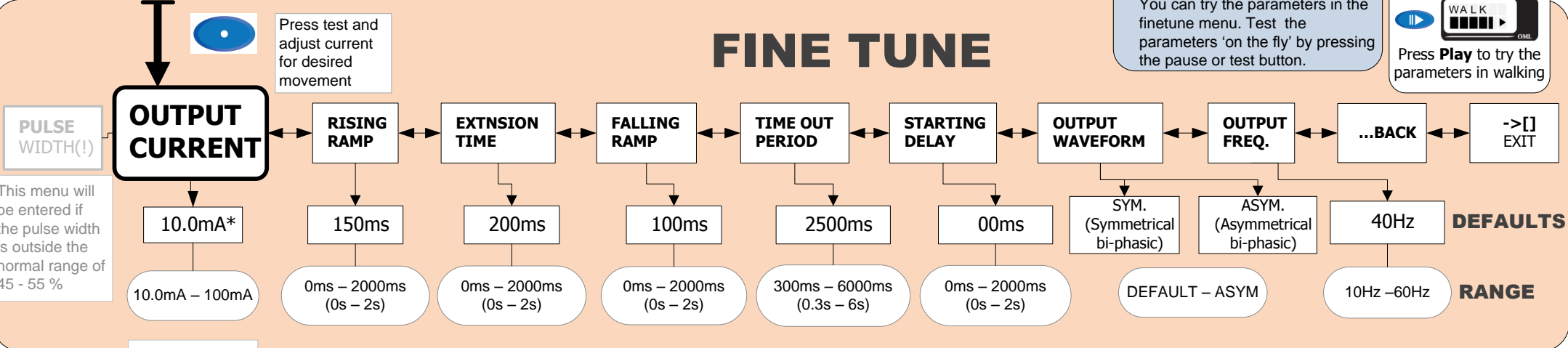


Press **Test** to try parameters

Press **Play** to try the parameters in walking

You can try the parameters in the finetune menu. Test the parameters 'on the fly' by pressing the pause or test button.


Stimulation envelope for Dropped Foot correction with heel rise trigger

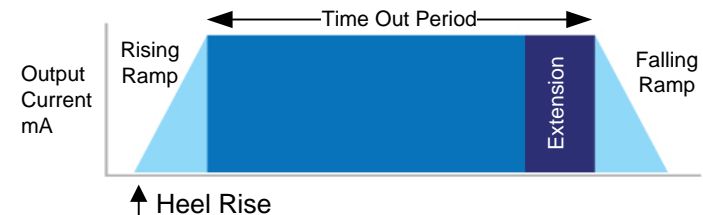


To enter setup mode:
(Pace must be in Pause mode)



Exercise mode and other settings


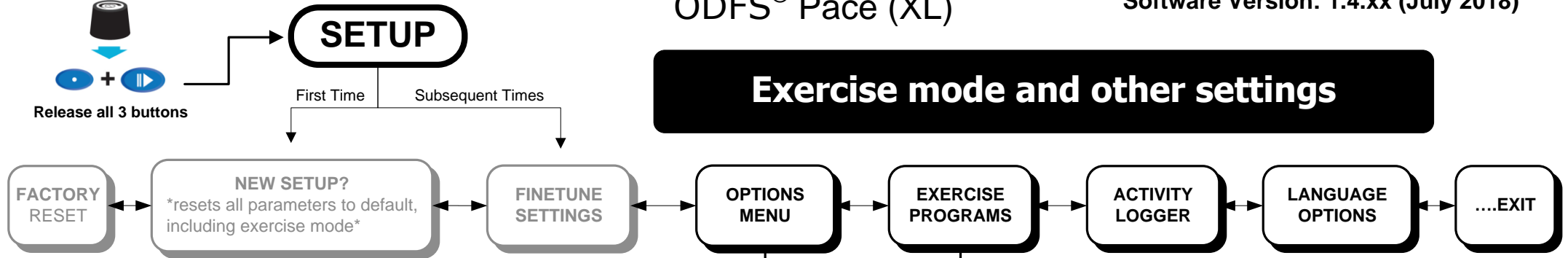


The **Test** button can be pressed to test the output parameters. Press the **Play** button to start / stop the exercise program.

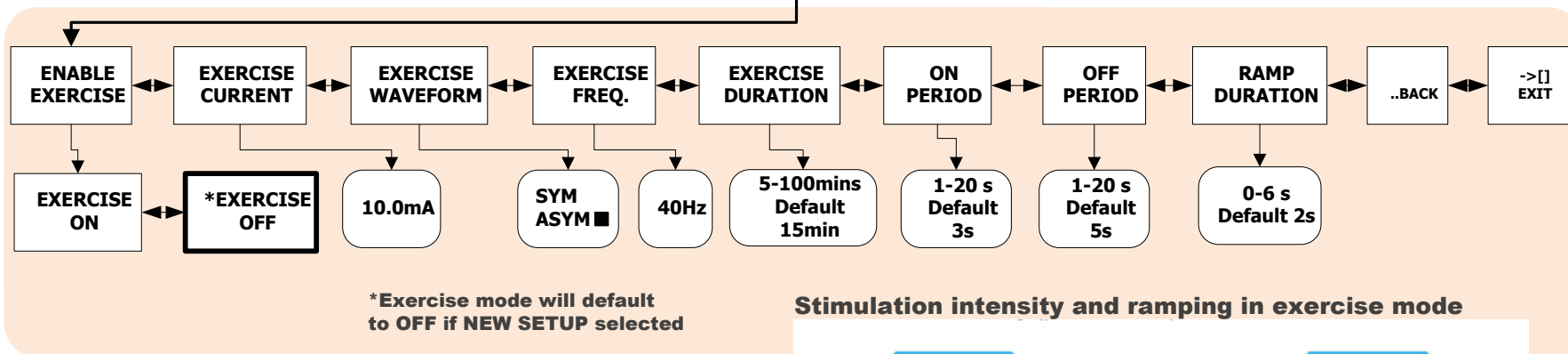

*Exercise mode will default to OFF if NEW SETUP selected

Stimulation intensity and ramping in exercise mode

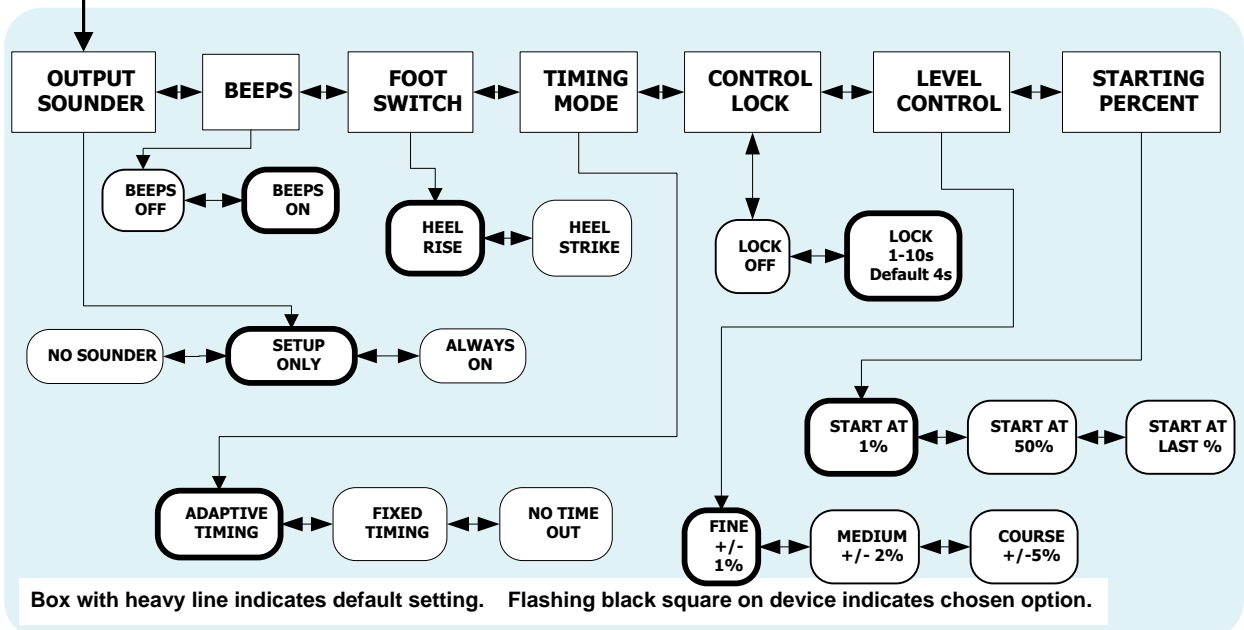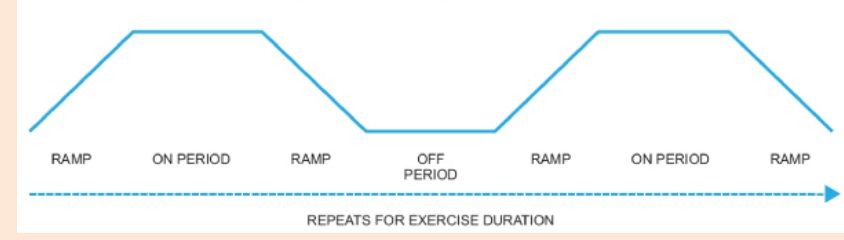


Box with heavy line indicates default setting. Flashing black square on device indicates chosen option.

Parameter View Mode

To enter parameter view mode:
With the device turned off



Navigate by twisting the control knob.
Press Pause to exit or hold the control knob to turn off

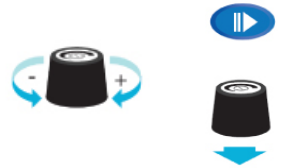

Use this to record the settings of the ODFS Pace